Name	LevMed Dedicated ECG Electrode Belt with Handles, D-Type With Screws Connector		
Model	ARh-DT-WS-LM		
REF	11061/h-LM		
SN	ARhWSL0616008		
	06/2016		

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Rx Only **Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician**

INTENDED USE

LevMed ECG Electrode Belt is a reusable accessory intended to be used with ECG devices for short-term 12-lead rest ECG recordings for adults and pediatrics.

SYMBOL LEGENDS

Caution, refer to instructions for use

Serial number

Catalogue number

Manufacturer

Date of manufacture

Non Sterile

WEEE- Waste of electrical and electronic equipment

WARNINGS AND CAUTIONS

- Do not use this device for any purpose other than the stated intended use.
- **Remove device before performing defibrillation.**
- LevMed ECG Belt is intended to be used for rest ECG recordings, make sure the patient is relaxed, lying still, on a flat surface.
- Stay away from devices that create strong electromagnetic fields.

CONTRAINDICATIONS

Do not use LevMed ECG belt on irritated or damaged skin.

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
INSTRUCTIONS FOR USE

1. DEVICE DESCRIPTION

LevMed reusable 12-lead ECG electrode belts enable simple, fast, accurate, and reliable assessments for both men and women, of all body shapes and sizes.

The ECG Belt **Dedicated** model is a system of ECG electrodes cast in a hyper-flexible silicone that is similar to the existing V1-V6 pre-cordial electrodes, LL, LA, RL, and RA. The Belt, including the conductors, is flexible and can be stretched to fit all body shapes and sizes, men and women alike; when fitted to the chest, it stretches to the required proportions, and the distance between the electrodes is automatically set. The Belt can be connected to a standard ECG with a suitable D-Type connector. The limb electrodes include 2 arm electrodes and a right leg electrode embedded in the belt, and a leadwire with a banana plug to be connected to a left leg electrode (clamp, disposable, or other). See Figure 1 below.

These instructions describe the application process of the following LevMed ECG Electrode Belt models:

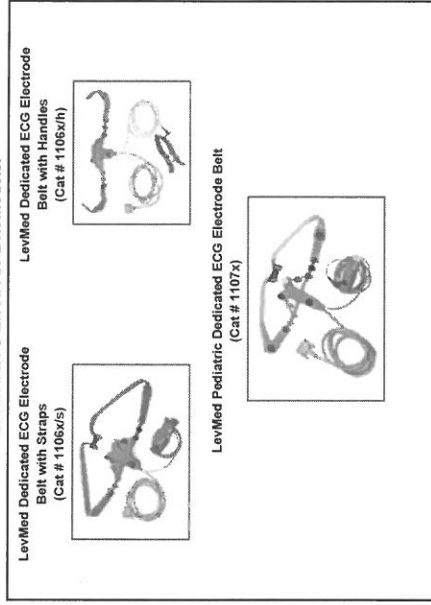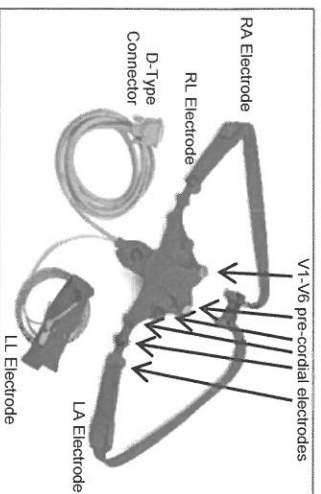
Figure 1: Dedicated Belt – Main Elements

2. GENERAL INSTRUCTIONS

- 2.1 Make sure that the ECG instrument you use is serviceable, and meets international safety standards.
- 2.2 Make sure you have a suitable connector for connecting the ECG instrument to the ECG Belt.
- 2.3 **It is recommended** to conduct the examination in quiet surroundings.
- 2.4 The patient should lie comfortably on a flat surface during the examination. It is not recommended to conduct the examination in a sitting position.
- 2.5 Wet the electrodes with water.
- 2.6 If there is fatty cream on the skin, remove it with alcohol. For very hairy people, conductive spray or gel may be used.

3. STEP 1 – BELT PREP:

- 3.1 Connect the cable to the ECG instrument. Make sure the connection is good.
- 3.2 Connect the stud plugs or banana plug to the clamp (Left Leg).
- 3.3 Wet the electrodes with water.

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4. STEP 2 – BELT APPLICATION

4.1 FOR DEDICATED BELT WITH STRAPS (1106x/s) & PEDIATRIC DEDICATED BELT (11007x)

- 4.1.1 For a cooperative patient:
 - a. Prepare the Straps: buckle the two sides of the straps according to general estimation of the required length. Note that it should be symmetrical, i.e. remaining flaps should be of the same length.
 - b. Put the patient's arms through the buckled straps and place them at the patient's sides.
 - c. Make further adjustment to the straps, if needed. See Figure 3 below.
- 4.1.2 For a less cooperative patient:
 - a. Prepare the Straps: adjust the buckles flaps according to estimation in a symmetrical way, without buckling first.
 - b. Patient should lay with arms at one's sides.
 - c. Place the belt on the patient's chest with V1 and V2 in place, on both sides of the sternum at the 4th intercostal space.
 - d. Place the straps beneath the patient's arms, and over the arms; buckle the Belt over V1 and V2.
 - e. Make further adjustment to the straps, if needed. See Figure 3 below.
- 4.1.3 V6 must reach the mid-axillary line. The other electrodes will then be set in their places.
- 4.1.4 Arrange the belt on the chest so that it is centered and stretched, with all the electrodes in contact with the skin.
- 4.1.5 Electrodes V1 and V2 should be positioned on both sides of the sternum at the 4th intercostal space. V6 must reach the mid-axillary line. The other electrodes will then be set in their places.
- 4.1.6 The buckle is placed over V1 and V2, pressing them towards the body.
- 4.1.7 Note: V1 and V2 electrodes may be bent towards the body if the patient has a sunken sternum.
- 4.1.8 Arms should be positioned at the side of the body. The patient should not strain the arms muscles, the belt is kept in position by the weight of the relaxed arms. See Figure 2.

Figure 2: Dedicated Belt with Straps Application

4.2 FOR DEDICATED BELT WITH HANDLES (1106x/h)

- 4.2.1 Patient's arms should be positioned at the side of the body.
- 4.2.2 Apply the Belt on the chest so that it is centered and stretched, with all the electrodes in contact with the skin. Electrodes V1 and V2 should be positioned on both sides of the sternum at the 4th intercostal space. V6 must reach the mid-axillary line. The other electrodes will then be set in their places.
- 4.2.3 V1 and V2 electrodes may be bent towards the body if the patient has a sunken sternum.
- 4.2.4 Right leg (ground) electrode is placed on the right side of the chest.
- 4.2.5 Fit the right handle of the Belt on the right arm and the left handle on the left arm. The handles should be placed over the mid-shaft of the humerus, between the elbow and the shoulder.
- 4.2.6 Arms should be positioned at the side of the body. The patient should not strain the arms muscles, the belt is kept in position by the weight of the relaxed arms. See figure 4.
- 4.2.7 Make sure the arm oval electrodes are in good contact with the patient's skin. See figure 3 below.

Figure 3: Dedicated Belt with Handles Application

5. STEP 3 – LIMB ELECTRODE PLACEMENT & ECG ASSESSMENT

- 5.1 Attach the LL clamp to the patient's left ankle. See Figure 3.
- 5.2 Activate the ECG instrument and conduct the examination.
- 5.3 **During the assessment, ensure the patient is relaxed and does not strain his/her muscles. The patient should try not to move or talk during the examination.**
- 5.4 When the examination has been completed, remove the Belt and the limb electrode.

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6. TROUBLESHOOTING AND MAINTENANCE

- In case of artifacts or lack of signal in one or more electrodes: make sure the electrodes are in contact with patient body. If necessary, re-wet the electrodes.
- Cleaning:**
Wipe the LevMed ECG Belt with disinfectant wipe after each patient. Use only disinfectants that are dedicated for use on medical devices.
CAUTION: Do not use disinfectants containing petrol.

WARRANTY

LevMed Ltd. warrants the ECG Electrode Belt to be free of defects in materials and workmanship for a period of 12 months from the date of purchase or 15 months from the date of production, whichever is shorter.
This warranty does not include or cover malfunctions caused by unreasonable use, noncompliance with user and maintenance instructions, or damage caused by unauthorized or unqualified repairs.

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