INSTALLATION

Optovist 2.0.010

SOMMAIRE

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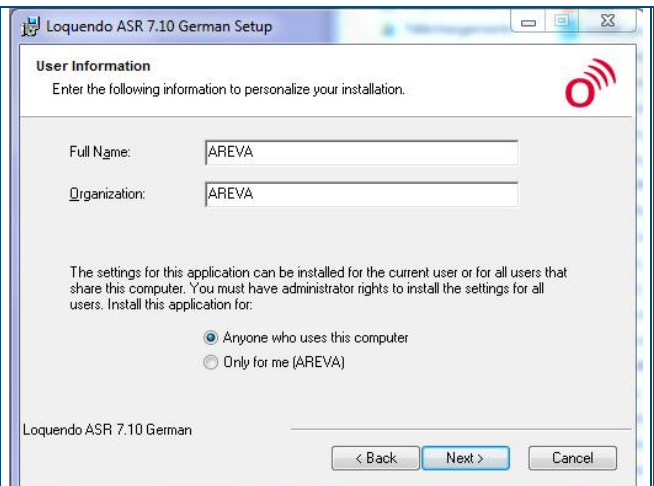
Loquendo ASR 7.10 German Setup

User Information
Enter the following information to personalize your installation.

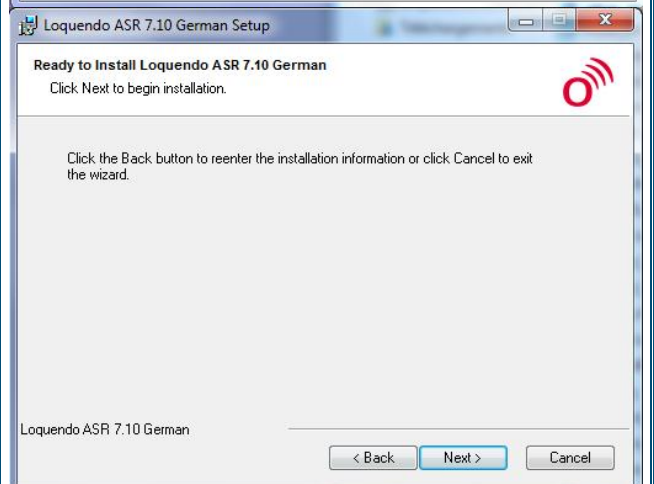
Full Name: AREVA
Organization: AREVA

The settings for this application can be installed for the current user or for all users that share this computer. You must have administrator rights to install the settings for all users. Install this application for:

☒ Anyone who uses this computer
☐ Only for me (AREVA)

Loquendo ASR 7.10 German

< Back Next > Cancel

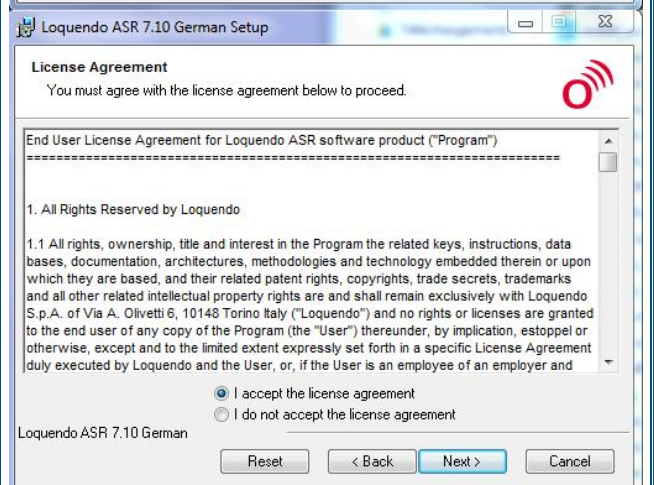
Loquendo ASR 7.10 German Setup

Ready to Install Loquendo ASR 7.10 German
Click Next to begin installation.

Click the Back button to reenter the installation information or click Cancel to exit the wizard.

Loquendo ASR 7.10 German

< Back Next > Cancel

Loquendo ASR 7.10 German Setup

License Agreement
You must agree with the license agreement below to proceed.

End User License Agreement for Loquendo ASR software product ("Program")

1. All Rights Reserved by Loquendo

1.1 All rights, ownership, title and interest in the Program the related keys, instructions, data bases, documentation, architectures, methodologies and technology embedded therein or upon which they are based, and their related patent rights, copyrights, trade secrets, trademarks and all other related intellectual property rights are and shall remain exclusively with Loquendo S.p.A. of Via A. Olivetti 6, 10148 Torino Italy ("Loquendo") and no rights or licenses are granted to the end user of any copy of the Program (the "User") thereunder, by implication, estoppel or otherwise, except and to the limited extent expressly set forth in a specific License Agreement duly executed by Loquendo and the User, or, if the User is an employee of an employer and

☒ I accept the license agreement
☐ I do not accept the license agreement

Loquendo ASR 7.10 German

Reset < Back Next > Cancel

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Loquendo ASR 7.10 German Setup

Loquendo ASR

Loquendo ASR 7.10 German has been successfully installed.

Click the Finish button to exit this installation.

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