Spirometry PC Software

Instructions for Use – English Instructions d'utilisation – Français Gebrauchsanweisung – Deutsch Instrucciones de uso – Español Instruções de utilização – Português Gebruiksaanwijzing – Nederlands

Contents

)
)
)
)
Ĺ
Ĺ
)
3
1
5
3
Ĺ
1
5
)
1
7
7
3
5
)
)

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Instructions for Use

I. Introduction - English

Thank you for choosing Spirometry PC Software. This instructions for use provides instructions to install and perform initial setup of the software. Once successfully installed, more comprehensive help is available through the Help menu when using the application.

The system is intended for use with paediatric (4 to 17 years) and adult (18 to 99 years) patients in hospitals, physician offices, laboratories and occupational health testing environments.

Intended Use

Spirometry PC Software is a PC application that is designed to collect, manage, and display data for Spirometry examinations. It is capable of interfacing with Vyaire Micro range devices, such as the Spiro USB, MicroLab and MicroLoop. It can perform live testing, upload of data from devices and can display these results graphically and allow for these results to be printed.

Contraindications

- Acute disorders affecting test performance (e.g. vomiting, nausea, vertigo)
- Recent eye surgery (increases in intraocular pressure during spirometry)
- Oral or facial pain exacerbated by a mouthpiece
- recent myocardial infarction
- post-operative thoracic surgery patients
- hyperventilation syndrome

Note: Extensive exhalation might lead to syncope

Warnings and Cautions

The following terms are used as follows in this manual

- **CAUTION**: Possibility of injury or serious damage.
- **WARNING**: conditions or practices that could result in personal injury.
- **Note:** Important information for avoiding damage to the instrument or facilitating operation of the instrument.

Note: Patients below 4 years of age may struggle to perform spirometry correctly and reproducibly.

Note: The device should only be used by trained and qualified personnel.

System Requirements

Spirometry PC software requires certain hardware and software components in order to run properly.

1. PC Requirements

An Windows-compatible PC is required, with hardware that meets or exceeds the following minimum requirements.

- Processor: 1 GHz or above
- RAM: 2 GB
- Free Disk Space: 200 MB
- Video: 800x600, 256 colours. It is recommended that a resolution of at least 1280x1024 is used to enable the full benefits of the multi-window interface.
- At least one free USB port. (An additional USB port will be required for USB enabled spirometers)

2. Operating System Requirements

Spirometry PC Software will run on the following operating systems:

- Windows 7 32-bit and 64-bit
- Windows 8.1 64-bit
- Windows 10 64-bit

Product Lifetime

The product lifetime of the software depends on the defined and released operating systems.

Installation

Before you begin, please ensure your computer meets the minimum system requirements and the user installing the software has administrative user rights.

- 1. Close any other applications that are running.
- 2. Insert the installation CD into your CD-Rom drive.
- 3. The setup program should launch automatically, displaying the welcome dialog box, click 'Next' to continue. If the setup program does not launch automatically use Windows explorer to manually select the CD drive then open the file named Spirometry PC Software.exe.
- 4. The license agreement will be displayed. Please read the complete document and ensure you understand fully before accepting the terms of the license. Click 'Next' to continue.
- The select destination location dialog will be displayed showing the directory where the Spirometry PC Software will be installed. The default location is C:\Program Files\Vyaire\Spirometry PC Software To change this location, select `Browse'.
- Click 'Next' to display the Select program manager group dialog. This will be the location where the Spirometry PC Software launch icon will be placed under the Start Menu. Either a new location can be entered or an existing location used.
- Click 'Next' to display the installation options dialog containing the following option:
 - Add Spirometry PC Software icon to the desktop.

- 8. Click 'Next' to start copying the files to your system.
- Once the files have finished copying to your system you will be advised that the Spirometry PC Software has been successfully installed and that it would be advisable to restart your PC before using the application. Click 'Finish' to complete the installation process.

Hardware Key

After installation, the below box is displayed.

Prior to using the Spirometry PC Software, you should insert the Spirometry PC Software hardware key into a free USB port on your PC. This will allow access to the Spirometry PC Software program. The hardware key provides freedom of use allowing the Spirometry PC to be installed onto numerous computers but can only be activated using the hardware key.

Please ensure that you keep the hardware key safe, it lost a replacement will need to be purchased.

Getting Started

The 'Settings' dialog will be displayed the first time you enter the Spirometry PC Software:

Settings include:

- Regional Settings Select the region and language settings.
- General Settings
 Select general settings such as height units, weight units, date format and date separator.
- Print Settings Select the settings for the printed reports.
- Spirometry Settings Select spirometry settings.
- Spirometry Indices Settings
 These options allow you to select which spirometry indices will be reported within various Spirometry PC Software windows and reports.

Please refer to chapter "Settings" in this manual for further details on the settings and for more information on changing the application settings.

As soon as you have entered or selected the appropriate settings, click on 'Save + Close'.

Once the settings screen is closed a default database will be created so testing can begin immediately. The default database is created in (C:\Program Files (x86)\Vyaire\Spirometry PC Software).

If a new database needs to be created click on 'Data' and then select the 'Create New Database' option. Enter the default password "Admin". Under 'File' the 'Create New Database' option will now be enabled.

Main Screen

From now on, the 'Main Screen' will be displayed by default after starting the Spirometry PC Software:

Toolbar Patient Test Test ID: 123456 New Search

The toolbar shows the currently selected patient (if available) and the current logged in user (if open database is secured). Here, it is also possible to select a new patient by clicking the 'New' button or search for an already existing patient by clicking the 'Search' button.

Main Tabs

🙆 Start Screen 🔍 Verify Calibration 🛛 Verform Test 🔍 Review Tests 🔊 Upload 🖏 Customise

The main screen provides the following main tabs:

Start Screen

The Start Screen tab is the default tab when the application is first run. It shows quick links to common tasks to get started with.

• Verify Calibration

The Verification tab allows you to perform calibration checks on connected devices (if the respective device supports a calibration check)

Perform Test

The Perform Test tab allows you to perform a live test with a connected device

Review Tests

The Review Test tab allows you to review examinations already performed on the selected patient. Additionally, an examination can be continued (e.g. perform post tests)

• Upload

The Upload tab allows you to upload patient and examination data from the connected device

Customise

The Customise tab allows the configuration of application settings as well as any configuration of the connected device.

Quick Links

The start screen provides 6 quick links to the following commonly used tasks:

Add patient

Enter a new patient into the current open database (see chapter 'Add Patient' for details)

• Search patient

Search for an existing patient in the current open database (see chapter 'Search Patient' for details)

• Verify Calibration

Verify the calibration on the connected device (see chapter 'Calibration Verification' for details)

New Examination

Start a new examination on the current selected patient (see chapter 'Perform Spirometry Testing' for details)

• Review Examination

Review examinations that have been previously performed on the current selected patient (see chapter 'Perform Spirometry Testing' for details)

• Upload

Perform upload from the connected device (see chapter `Data Upload' for details)

Function Buttons

The functionality of the function buttons change according to the currently displayed tab. The buttons can be activated by clicking on them with the mouse or pressing the associated function key on the keyboard (i.e. **F2**, **F3**...)

Add Patient

In order to add a new patient to the database, click on the 'Add new Patient' quick link in the Start Screen tab.

Blew Alternatively, click the 'New' button in the toolbar.

The Add Patient form appears:

ld Patient					
Patient Detail	s				
ID		*			
First Name		Smoking	Not Specified	~	
Surname		Dysphoea	0 🗄		
DOR	DD MM YYYY	Arre	10 years 0 months		
Candan	205 15 20075*	Ange .	To years o montais		
Gender		Occupation Redu Mass			
Height (cm)		BOUY Mass			
weight (kg)	0.0	Referred By	I		
Ethnic Origin	Caucasian 💌 *				
* Indicates	a mandatory field				
Clinical Detail	s				
Spiromet	try Details 🛛 🕏 MVV Details	Medication	2 Patient History	-	
Predicted S	et			Factor (%	
Europe (ECC	CS, Cogswell, Solymar, Zapletal)		- 🔺	100 🛨	
Warning: Th	e patient's height is outside the v	alid range for the p	edicted set, and is trun	cated when calculating	
				F2 P3 F3	1 5° X
				Save + New Save	+ Close Cancel

It is possible to enter the following patient demographic details:

• ID*

Enter a unique alphanumeric identifier for the patient. An ID must be given and must be unique within the current selected database.

• Alias ID

Extra optional ID field (used for some older devices). Any ID given must be unique and must not match any ID within the current selected database.

• Surname

• First Name

DOB (Date of Birth)*

Date format can be set in the 'General Settings' tab of the Main Screen (see chapter 'Settings' for details).

• Gender*

• Smoking

Enter the patient's smoking status

• Height*

Enter the patient's height. Measurement units can be set in the 'General Settings' tab of the Main Screen (see chapter 'Settings' for details).

• Weight

Enter the patient's weight. Measurement units can be set in the 'General Settings' tab of the Main Screen (see chapter 'Settings' for details).

• Ethnic Origin*

• Factor

Enter the patient's ethnic correction factor. This is a percentage to factor the patient's predicted values by.

• Dyspnoea

Enter patient's dyspnoea score (a numeric value 0 - 20 that indicates how ill the patient is).

- Occupation
- Referred By

Enter the name of the referring doctor.

• Age

The patient's age is calculated automatically from the entered DOB.

• BMI (Body Mass Index)

The patient's BMI is calculated automatically from the respective values entered.

Fields marked with a star ('*') are mandatory, fields not marked with a star are optional.

It is also possible to enter following clinical details for each patient:

• Patient History

Click the Patient History tab and enter details of the patient's medical history.

• Medication

Click the Medication tab and enter details of any medication the patient is taking.

• Spirometry/MVV Specific Details

Click the respective tab for the test type you wish to enter details for.

Click 'Save + New' to save the new patient and clear the form. This allows you to enter multiple patients without having to re-open the Add Patient form each time.

Click `Save + Close' to save the new patient and close the Add Patient form.

Click 'Cancel' to close the Add Patient form without saving the new patient.

Saved patient data can be edited at any time. See chapter 'Modify Patient' for details.

Search Patient

In order to search for an existing patient in the database, click 'Search Patient' in the Start Screen tab.

Search Alternatively, click the 'Search' button in the toolbar.

The Patient Search form appears:

Search Basic A	dvanced							
Su	urname DOB	YYYY MM (First Name	YYYY MM DD	ID Clear Search			
Patients							_	_
Ame:	Crouch, A CA57	lec			DOB:	1957/10/28	Age:	5
Ame: ID:	Denton, J DJ81	ames			DOB:	1981/12/07	Age:	
Ame:	Smith, Joh JS83	In			DOB:	1983/02/06	Age:	
A Name: ID:	Smith, Joh SJ83	ท			DOB:	1983/02/06	Age:	
			ſ	Help				
Total Patient	its: 4 (Filtere	:d: 4)		Search matching patients, wh fit the values 'first name', 'sur	ich can be combined with the select name', 'date of birth' and 'gender' o	ted patient. It of the patients	must	b
F3	F4			F6	F	Folart	F9	R

The Patient Search form allows you to search for a specific patient and select this patient for review or test.

There are two methods by which a patient can be searched: Basic search or Advanced search.

Basic Patient Search

This search method allows you to search on the patient's Surname, First Name, ID, DOB (Date of Birth) and Exam Date by entering the respective data in the appropriate field. Any combination of these search parameters can be defined.

The displayed list of patients will automatically be updated according to the search parameters entered.

To clear the entered search parameters, click the 'Clear Search' button.

The list can be sorted by the patient's ID, Surname or First Name. The patient field that you want to sort the list by can be changed by right clicking the mouse while the cursor is on the list. A pop-up context menu will then be displayed. Click the 'Sort by' menu item and choose the field you want to sort the list by from the resulting menu. The field that the list is currently sorted by is checked.

Highlight the required patient in the list by left clicking with the mouse.

Click 'Select' to select the highlighted patient and close the Patient Search form

Click 'Cancel' to close the Patient Search form without selecting a patient.

Advanced Patient Search

This search method allows for a more comprehensive and complex search to be performed on the database. This is useful if you have a particularly large database with many patient records. The advanced patient search also allows for you to search for patients with abnormal test results.

To add a new patient to the database, click the 'Add Patient' button and proceed as described in the 'Add Patient' chapter.

Merge Patients

If a patient has been entered into the database with the incorrect ID or uploaded from the spirometer with the incorrect ID, it is possible to merge two patients to have only one entry with the correct ID.

To perform this task search for the patient that you wish to be the sole record with the correct ID you wish to use in future. A list of possible matching patients will be displayed in the search box.

- Only those patients with matching demographic details will be shown in the list
- If a patient is not listed that you know is the same patient then you will need to change the patient demographics so that it appears in the search list.

Press the 'Merge To Patient' button and the examinations for the patient will be merged to the original patient you have selected creating one record for that patient.

💀 Merge To Patient					• X
ID \$383	Surname	Smith	First Name	John	
	DOB	1983/02/06	Gender	Male	
Patients					
Name: Smith, John ID: JS83				DOB: 1983/02/06	Age: 34
Help Merge selected patient to thi	s patient.				
	<u> </u>				
F8 Merge To Patient					F9 Cancel

Modify Patient

It is possible to modify the demographic details of an existing patient.

If the respective patient is already selected and is displayed in the toolbar, click the patient's name in the toolbar.

Alternatively, search and select the respective patient as described in the 'Search Patient' chapter and click the 'Modify Patient' button in the Patient Search form.

The Modify Patient form is displayed:

Modify Patient					×	
Patient Detail	5					
ID	123456	*				
First Name	John		Smoking	Not Specified	•	
Surname	Smith		Dyspnoea	0 ÷		
DOB	DD MM YYYY 17 1 1960 *		Age	57 years 0 months		
Gender	Male 💌 \star		Occupation			
Height (cm)	180 🛨 🗙		Body Mass	29,3		
Weight (kg)	95.0 ÷		Referred By		•	
Ethnic Origin	Caucasian 💌 \star					
* Indicates a	a mandatory field					
Clinical Details	5					
🔗 Spiromet	ry Details 💎 MVV Details	E	Medication	2 Patient History	-	
Predicted S	et				Factor (%	
Europe (ECC	S, Cogswell, Solymar, Zapletal)			– ©	100 ÷	
		_				
F ⁴ Change ID						

Make the desired modifications.

Click the 'Save + Close' button to save the modified patient details and close the Modify Patient form.

Click the 'Cancel' button to close the Modify Patient form without saving the modified patient details.

All fields as described in the 'Add Patient' chapter can be modified, except for the patient's unique ID field by default.

To enable the changing of the patient's unique ID, click 'Change ID' and 'Yes' when presented with the confirmation message box.

NOTE: It is not recommended to change a patient's unique ID!

Calibration Verification

In order to verify the calibration of the connected device, click the 'Verify Calibration' Quick Link or change directly to the 'Verify Calibration' tab on the start screen. Following window appears:

Click the 'Verify Calibration' button to invoke the start of a device search. The device search effectively scans the PC's comms/USB ports for any attached device.

On finding a device, the screen will change to the main verify calibration test screen:

File Data Patient The	e Customer Help) New Search			CareFusior
F1 ? Heb	Diant Screen Verify Calibration Enter User Name User Name	Unit For Calibration Verification Unit For Calibration Verification Unit Nam Device Serial Numb	Upload Upload SpealUS8 2 Come P	or USB	
	Select Syringe Size Syringe Size 3 Litres	Select Manoeurre C Explatory Only G Explatory and Inco	eatory	ation Verification Requirements C Simple @ AT\$/\$/\$/R\$	
	How(Vs) 12 10 6 6 4 2		Band (low 0 - 0.91/s Test 1 EXP) Walting	IUEAL MEA
¹⁷		e e 10 volume@	x]		
Ext		View History Start Tes	t Exit Verity Calibration	Next Band	ot

User Name

A user name entry is required before a calibration test can proceed. Select a user name by clicking the drop down menu or enter a new user name manually.

Unit For Calibration Verification

• Unit Name

This field should have the device name automatically entered and not allow any changes.

• Device Serial Number

If a device serial number can automatically retrieved from the device, it will automatically be entered and not allow any changes. However, if it is not possible to retrieve the device serial number, one can be entered manually. A device serial number entry is required before a calibration test can proceed.

• Comm Port

The port at which the device has been located should be placed here automatically. Once automatically entered, this field cannot be altered.

Syringe Size

The syringe size automatically defaults to 3 litres. However, clicking on the drop down menu allows a selection between a 1-6 litre syringe.

Select Manoeuvre

This selection depends on the device capabilities.

Select Verify Calibration Requirements

Provides a choice of calibration tests between a simple requirement calibration test or an ATS/ERS Requirement calibration test.

• ATS/ERS Requirement Calibration Test

The ATS/ERS standardisation of Lung Function Testing requirements recommend that when verifying a calibration 3 waveforms are performed at 3 different flow rates (low, medium and high). This is to give a range of flow variability (between 0.5 and 12 litres) with the individual flow bands as follows:

- Low flow (0 0.9 L/s)
- Medium flow (1.6 4.5 L/s)
- High flow (7 12 L/s)

• Simple Requirement

A simple requirement test comprising of just performing one syringe stroke at the users choice of flow rate.

Flow Volume Graph

A flow volume graph is used to display the calibration test manoeuvres in real time:

Results table

All results are automatically displayed within the results table after each manoeuvre has been performed:

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• Stage

Current band limits with current test and whether it is an expiratory or inspiratory test (with ATS/ERS requirement test) / Current test and whether it is an expiratory or inspiratory test (with Simple Requirement test)

• Status

Test status:

- Waiting: Waiting for a test to begin
- Testing: a test is under way
- Skipped (ATS/ERS requirement only): band has been skipped
- Passed: test manoeuvre has passed
- Failed: test manoeuvre has failed

- Ideal Ideal value to base the percent error on
- Meas Actual value calculated during the manoeuvre

• Error

The percentage error of the calibration test based on the percent difference between the actual value and the ideal value.

To begin the calibration test, press 'Start Test'. A message will be displayed indicating that the syringe should be connected to the unit with the minimum of adapters and the plunger should be fully pulled out. Click 'OK' to exit this message.

Pump evenly and avoid banging the syringe against the stop. Click 'Next Band' to skip current ATS/ERS requirement band (only displayed when performing an ATS/ERS requirement calibration test). Wait for the end of calibration and results to be displayed.

An ATS/ERS requirement test is passed if the volume measured is within the expected rate (only a +/- 3 % deviation to the expected value is allowed). A minimum of one test per band needs to be passed fully.

Click 'Print Calibration' to print a calibration report. The one page colour report includes the user name, spirometer model, serial number, calibration syringe size, calibration manoeuvre type, results (ideal volume, measured volume, percent error), whether the calibration check passed or failed, signature line and the date and time at which the report was printed.

Click 'Finish Test' to finish the current calibration verification regardless of whether the test is complete and revert back to the start screen.

Click 'View History' to view the devices calibration verification history. The history is based on a device serial number which needs to be entered into the correct field before the history file can be viewed. On entering the required device serial number, clicking the 'View History' button automatically displays selected parameters (user name, date and time of calibration, selected manoeuvre type, syringe size, whether the device passed the calibration check or failed, if it failed, the reason why).

Perform a Test

To start a new examination, select the patient you want to perform the examination on (see chapter 'Search Patient' for details on how to select an existing patient or chapter 'Add a Patient' for details on how to add a new patient). Click the 'New Examination for' Quick Link or change directly to the 'Perform Test' tab on the start screen. Following window appears:

If no calibration verification has been performed that day, a warning message appears:

Warning		×
1	A calibration verification has not been made today. Would you like to perform a calibration verification now?	
F2 Yes	F ³ No	

Click 'Yes' to perform a calibration verification now (details see chapter 'Calibration Verification') or click 'No' to continue without performing a calibration verification.

Following window appears:

New Exam				×			
Patient Details							
Height (cm) 180 📑	ВМІ	29.3				
Weight (kg) 95.0 🗄	Occupation					
		Smoking	Not Specified	•			
Dyspnoea	0 🗄	Referred By		•			
Predicted S	Predicted Set Europe (ECCS, Cogswell, Solymar, Zapletal)						
Medication							
Please ensure that the Spirometer is connected to the computer. Switch on the Spirometer and put into PC communication mode if applicable (refer to user manual).							

Here, the patient details saved for the current selected patient that the examination will be performed on are displayed. Before the new examination is started, any of these fields can be modified as some of the patient details might have changed (e.g. weight) since the patient record was last added or modified.

Please check the correct patient details prior to starting the examination and, if necessary, perform amendments.

Note: Incorrect patient data produce incorrect predicted values!

The Spirometry module allows for forced and relaxed spirometry to be performed.

Click the 'Relaxed Base' button to start the examination with a relaxed test. (The type of relaxed test depends on the Relaxed Mode selected in the Spirometry Settings. Please see chapter 'Settings' for details.)

Click the 'Forced Base' button to start the examination with a forced test

Note: Please ensure that the device is connected to the PC, is powered and is in PC communications mode.

When a new test is started within an examination, a customisable Spirometry Perform Test Panel is displayed. The following windows can be displayed when performing a new live test:

• Analysis Window

Shows analysis on the blow that has just been performed and compares that blow with the best blow performed so far. It will also display the blow quality and whether reproducibility criteria has been met yet or not.

- Live FVL (Forced tests only) Flow Volume Loop for the current live blow during the forced spirometry manoeuvre. It also shows the predicted area or line (depending on the settings)
- Best FVL (Forced tests only) Shows the Flow Volume Loop for the current live blow during the forced spirometry manoeuvre and the Flow Volume Loops for all the best forced tests of the current examination. It also shows the predicted area or line (depending on the settings)

• All FVL (Forced tests only)

Shows the Flow Volume Loop for the current live blow during the forced spirometry manoeuvre and the Flow Volume Loops for all other forced tests of the current examination. It also shows the predicted area or line (depending on the settings)

• Live VTG

Shows the Volume Time Graph for the current live blow during the spirometry manoeuvre. It also shows the predicted area or line (depending on the settings)

• Best VTG

Shows the Volume Time Graph for the current live blow during the spirometry manoeuvre and the Volume Time Loops for all the best tests of the current examination. It also shows the predicted area or line (depending on the settings)

• All VTG

Shows the Volume Time Graph for the current live blow during the spirometry manoeuvre and the Volume Time Loops for all the other tests of the current examination. It also shows the predicted area or line (depending on the settings)

• Volume Bar (Relaxed tests only)

Shows a bar chart indicating the expired/inspired volume during the relaxed spirometry manoeuvre. It also shows the minimum and maximum predicted volume and the best volume achieved in previous stages of the examination.

• Incentives (Forced tests only)

Shows the incentive animation during the forced spirometry manoeuvre that represents whether the patient is expiring or inspiring and indicates whether the patient has reached their predicted or not. It is possible to select from multiple incentive animations.

• Examination Notes

Shows textual notes on the examination and allows for the notes to be modified.

Review Tests

To review the examinations that have been previously performed on a patient, select the patient whose examinations you want to review (see chapter 'Search Patient' for details on how to select an existing patient or chapter 'Add a Patient' for details on how to add a new patient). Click the 'Review Examinations for' Quick Link or change directly to the 'Review Tests' tab on the start screen. Following window appears:

The **Examinations List** lists all examinations that have been performed on the current selected patient.

The **Test List** lists all of the individual tests that were performed as a part of the selected examination.

The Review Tests Tab also shows a customisable display of different windows that show information on the current selected examination and the current individually selected test:

- Selected FVL (Forced test results only) Shows the Flow Volume Loop for the test that is highlighted in the Test List
- **Best FVL** (Forced test results only) Shows the Flow Volume Loops for all the best forced tests of the selected examination
- All FVL (Forced test results only) Shows the Flow Volume Loops for all the forced tests of the selected examination

• Selected VTG

Shows the Volume Time Graph for the test that is highlighted in the Test List

• Best VTG

Shows the Volume Time Graph for all the best tests of the selected examination

• All VTG

Shows the Volume Time Graph for all the tests of the selected examination

• Analysis

Shows an analysis of the selected examination. It displays interpretation bars, textual interpretation, predicted set used to calculate predicted values for this examination and whether the examination meets the reproducibility criteria

• Summary Results

Shows a summary of the results for the selected examination. Shows the values for the selected indices for the best base and post tests. It also shows the predicted value (if available) for the indices and compares the actual value to this

Results

Shows the results for the selected test. It shows the values for the selected indices and values for the best tests, predicted values and compares the actual results to these values

• Patient Summary

Shows a summary of the patient's demographic details at the time of the examination

• Examination Notes

Shows some textual notes on the examination and allows for these notes to be modified

• **Trend** (Forced test results only) Shows a trend graph that plots selected indices from all the patient's spirometry examinations.

Left click on the desired examination item in the Examinations List to select an examination to review.

To select an individual test to review, left click on the test item in the Test List.
Continue a Test

To continue an existing examination (i.e. perform a post test), select the respective patient you want to continue the examination on. Open the Review Test Tab (see chapter 'Review Tests') and select the examination you want to continue in the Examinations List.

Click on the 'Continue Exam' button.

Printing

To print an examination, select the respective examination within the Examinations Lists on the Review Tests Tab (see chapter 'Review Tests'). Click the 'Print' button on the left of the main screen or click 'Print' on the File menu. To preview the printed report, click 'Print Preview' on the File menu in the main tool bar.

It is also possible to print all examinations performed today or between to dates. For this, hover the 'Batch Print' menu item on the File menu in the main tool bar. Select 'Today's Examinations' on the resulting popup menu or, respectively, click 'Examinations Between Dates' on the resulting popup menu and enter a 'From' as well as a 'To' date in the displayed Print Exams Between Dates form and confirm with 'Ok'.

For information on print settings see chapter 'Settings'.

Settings

Click the 'Customise' tab in the main screen to modify the settings made the first time the Spirometry PC Software was entered (see chapter 'Getting Started') and to make further settings. Following screen appears:

Select Region

Click 'Select Region' in the 'Customise' tab to select region and language settings. Following screen appears:

The countries and regions supported are shown in dark blue on the map, unsupported in grey. Select the required country/region by left clicking it on the region map.

Alternatively, click the 'Arrow down' in the 'Country (culture)' section and select the required country/region from the drop-down menu.

The appropriate supported language for the selected region will automatically be loaded in the 'Language' combo box. Clicking the 'Arrow down' in the language combo box will open a drop down menu where the required language can be selected from.

By default, the region and language currently setup in Windows is selected unless the software does not support that particular region and language in which case United Kingdom, English is selected by default.

Click 'Save' to save the new settings. The window will not be closed.

Click 'Save + Close' to save the new settings and close the window.

Click 'Cancel' to close the window without saving the new settings.

General Settings

Click 'General Settings' in the 'Customise' tab to select general application settings. Following screen appears:

% Settings	_ 🗆 🗙
🚱 Regional Settings 🛛 🗞 General Settings 🛛 🏷 Print Settings 🛛 🖉 Spirometry Settings 👘 🖏 Spirometry Indices Setti	₹
Units of Measure	
Height will be measured in: Weight will be measured in:	
● cm C inches ● kg C lbs	
Date Display Options	
Dates shall be displayed in the format:	
Dates will be displayed using the separating character:	
0/ 0- 0.	
Backup	
Perform automatic backup of database on application startup	
New Exam	
Start a new Spirometry exam	
C Start a new MVV exam	
C Prompt for which exam type to perform	
Configuration:	
Standard C Lite	
FS Save For Close	F9 Cancel

• Units of Measure

Select the required units for height and weight

• Date Display Options

Select the format the date should be displayed in and the separating character.

• Backup

If the box is checked, an automatic backup of the database is performed on application startup.

• New Exam

Select if a new spirometry exam or a MVV exam is started when the 'New Exam' button in the main screen is clicked or if a message box appears asking which exam type should be performed

Configuration

Select Standard or Lite Configuration

Click `Save' to save the changes. The window will not be closed.

 $\mbox{Click `Save + Close' to save the changes and close the window.$

Click 'Cancel' to close the window without saving the changes.

Print Settings

Click 'Print Settings' in the 'Customise' tab to set the options for printed reports. Following screen appears:

% Settings	_ 🗆 🗙
🕼 Regional Settings 🛛 🗞 General Settings 🛛 😓 Print Settings 📝 Spirometry Settings 🛛 🖓 Spirometry Indices Setti	
Print Header Printout Tite Printout Description Printout Description	
Print Settings Automatic PDF Export	
Format in Black and White Export all new/modified exams as PDF's to the following Page Margins (cm)	
Top (Frst Page) Left 1.0 ² / ₂ 1.0 ² / ₂ Top (Subsequent Pages) Right 1.0 ² / ₂ 1.0 ² / ₂ Bottom .	
1,0 Browse	
Print Options Spirometry Print Option: Basic Change	
FS Save Save Close	²⁹ Cancel

• Print Header

Set the title that will be printed at the top of all printed reports. Two title lines are available to be used, e.g. to display name and address of the clinic/hospital at the top of the printed reports

• Print Settings

Check the 'Format in Black and White' to print the report in black and white and set the page margins as required.

• Automatic PDF-Report

Check the respective box if all new or modified exams should automatically be exported as PDFs. Click 'Browse' to select the folder the PDF report should be saved to.

• Print Options

Here, print options can be set for each test type available to the software. Click on the 'Change' button next to the respective test type to select from the following print options:

- **Basic**: printout with minimal information. In most cases, a Basic printed report should fit on a single page.
- **Intermediate**: slightly more advanced printout than the simple report, though not all information is printed
- Advanced: most advanced printed reports, contain all possible information
- Custom: define your own print options for the respective test type. For this, click 'Edit' to the right of the combo box.

Click `Save' to save the changes. The window will not be closed.

Click `Save + Close' to save the changes and close the window.

Click 'Cancel' to close the window without saving the changes.

Spirometry Settings

Click 'Spirometry Settings' in the 'Customise' tab to select spirometry options. Following screen appears:

• Preset Configuration

Set a specific preset configuration for spirometry. The available preset configurations are:

 ATS/ERS Criteria 2005: this preset configuration will ensure that the application is fully compliant with the ATS/ERS Criteria 2005

- ATS/ERS Criteria 2005 (no short blow): this preset configuration will ensure that the application is compliant with the ATS/ERS Criteria 2005, but will not show the 'short blow' warning when a blow is less than 6 seconds.
- NICE: this preset configuration will ensure that the application is compliant with the NICE criteria
- General
 - **Show Grids**: Check the box if grid lines should be shown on the Volume Time / Flow Volume Loops
 - **Show Lung Age**: Check the box if the patient's lung age should be shown in the results window
 - Show Short Blow Warning: Check the box if the 'short blow' quality warning should occur when a blow's expiratory manoeuvre is less than 6 seconds
 - Hide FEV1/FVC % Predicted: Check the box if the FEV1/FVC% predicted values should not be shown when displaying the results
 - Verify Calibration Reminder: Set the periodic verify calibration reminder.

Available options:

- **Off:** No verify calibration reminder will ever be displayed
- **Daily**: The verify calibration reminder will occur on a daily basis
- **Weekly**: The verify calibration reminder will occur on a weekly basis
- **Maximum Post Tests:** Select the maximum number of post tests that can be performed in a single examination
- Auto Exclude Bad Live Tests: Check the box if bad tests (neither 'good blow' nor ' short blow' quality) should automatically be excluded.

- **Peak Flow Units**: Select the flow units for the Peak Flow indice (I/s or I/min)
- **Flow Type**: Select a flow type (Expiratory + Inspiratory or Expiratory Only) that will govern the display of the graphs that are displayed in both live mode and when reviewing existing spirometry examinations.
- **Relaxed Mode**: Select the mode for performing relaxed spirometry tests. Available options:

EVC: Performing a relaxed test only requires a single expiratory manoeuvre

IVC: Performing a relaxed test only requires a single inspiratory manoeuvre

Tidal: Performing a relaxed test requires a full tidal breathing manoeuvre

Click 'Spirometry Indices' to directly open the Spirometry Indices Settings window. Details see section 'Spirometry Indices Settings' below.

Click 'Colour Scheme' to view/edit the various colours used when displaying post stages.

- Advanced
 - Interpretation
 - Reproducibility Criteria
 - IC Summary Calculation: Available Options: IC from best test Average IC
 - Acceptable inhale/exhale ratio

- Forced Best Criteria:

Available Options: FEV1 + FVC FVC FEV1 PEF Individual Best

- Relaxed Best Criteria: Available Options: FEV1 + FVC FVC

- Forced Individual Best Quality: Available Options: None NLHEP QC Grade
- **Trend**: Tailor the display of the spirometry trend graph. Available options are:

Absolute value: The trend graph will plot absolute indice values

% **Predicted:** The trend graph will plot indice values as a percentage of the predicted value

Z-Score: The trend graph will plot indice values as a Z-Score of the predicted value

Tick the indices that should be shown on the trend graph. Available options: FVC

PEF FEV1 VC FEF 25-75 FEV1/FVC

- Predicted Set
 - Default Predicted Set: Define the default predicted sets. Predicted sets are stored for each individual patient and examination. Define Predicted set that will be used for each patient unless manually modified.
 - Forced Flow Volume Predicted Display: Select the predicted display method on Flow Volume Loops for forced spirometry manoeuvres. Available options are:
 Area: Shaded predicted areas will be displayed
 Line: Predicted line will be displayed
 None: No predicted display
 - Forced Volume Time Predicted Display: Select the predicted display method on Volume Time Loops for forced spirometry manoeuvres. Available options are:
 Area: Shaded predicted areas will be displayed
 Line: Predicted line will be displayed
 None: No predicted display
- **Incentive Target**: Define the incentive target for the first test and the subsequent tests.

Click `Save' to save the changes. The window will not be closed.

 $\mbox{Click `Save + Close' to save the changes and close the window.$

Click 'Cancel' to close the window without saving the changes.

Spirometry Indices Settings

Click 'Spirometry Indices Settings' to select which spirometry indices should be reported within the results screens and on the printed reports. Following window appears:

Select the indices to be displayed by checking the respective boxes.

Note: The number of selected indices is not restricted. However, please keep in mind that the same indices will be printed out in the report as well. Selecting too many indices will make the report unnecessarily long.

Overview Key Forced Spirometry Indices:

Overview Key Relaxed Spirometry Indices:

Click `Save' to save the changes. The window will not be closed.

Click 'Save + Close' to save the changes and close the window.

Click 'Cancel' to close the window without saving the changes.

MVV Settings

Click 'MVV Settings' in the 'Customise' tab to change MVV settings. Following screen appears:

🛸 Settings	_ 🗆 🗙
🕞 Regional Settings 🛛 🗞 General Settings 🛛 😓 Print Settings 🛛 🖉 Spirometry Settings 🗍 🖓 Spirometry Indices Setti	₹
Preset Configuration	
ATS/ERS Criteria 2005	
□ General	
Show Grids	
Maximum Post Tests Auto Exclude Bad Live Tests MVV Indices Colour Scheme	
∼ Advanced	
Reproducibility Criteria	
ATS/ERS Criteria [2005]	
Predicted Set	
Default Predicted Set	
USA (Polgar & Promadhat, Kory)	
FS Save For Close	F9 Cancel

• Preset Configuration

Set a specific preset configuration for MVV. The available preset configurations are:

- ATS/ERS Criteria 2005: this preset configuration will ensure that the application is fully compliant with the ATS/ERS Criteria 2005

- General
 - **Show Grids**: Check the box if grid lines should be shown on the Volume Time graphs
 - **Maximum Post Tests:** Select the maximum number of post tests that can be performed in a single examination
 - Auto Exclude Bad Live Tests: Check the box if bad tests (no 'good blow' quality) should automatically be excluded.

Click 'MVV Indices' to directly open the MVV Indices Settings window. Details see section 'MVV Indices Settings' below.

Click 'Colour Scheme' to view/edit the various colours used when displaying post stages.

Advanced

Select the reproducibility criteria that will be applied to new and existing examinations. Available options are:

- ATS/ERS Criteria [2005]: this preset configuration will ensure that the application is fully compliant with the ATS/ERS Criteria 2005
- Predicted Set
 - Default Predicted Set: Define a default predicted set. The predicted set is stored for each individual patient and examination. Here, it is possible to define a default predicted set that each new patient will use unless manually modified.

MVV Indices Settings

Click 'MVV Indices Settings' in the 'Customise' tab to select which MVV indices should be reported within the summary results screen and on the printed reports. Following window appears:

Select the indices to be displayed by checking the respective boxes.

Note: The number of selected indices is not restricted. However, please keep in mind that the same indices will be printed out in the report as well.

Click `Save' to save the changes. The window will not be closed.

Click `Save + Close' to save the changes and close the window.

Click 'Cancel' to close the window without saving the changes.

Device Configuration

Click 'Device Configuration' in the 'Customise' tab to perform a device configuration. A device search will be automatically performed. As soon as the connected device is found, you will be prompted with a form specially designed for performing configuration for your device.

Note: Only certain devices support device configuration.

Note: Please ensure that the device is connected to the PC, powered and in PC communications mode.

Today List

Click 'Today List' in the 'Customise' tab to create and download a list of patients to be tested today. This list can then be downloaded to a device ready for the day's appointments.

The 'Today List' form appears which is similar to the Patient Search form (see chapter 'Search Patient'). A patient can be searched for and selected the same way as on the Patient Search form. However, on the Today List form, multiple patient rows can be selected within the patient list.

Beneath the patient list, the Today List is located which lists all patients selected for the today list. This list is empty by default.

To **add a patient to the Today List**, highlight the respective patient in the patient list and click the 'Add' button. Following this, the patient will automatically appear in the Today List.

To **remove a patient from the Today List**, highlight the respective patient in the Today List and click the 'Remove' button.

To **remove all patients from the Today List**, click the 'Clear Today List' button.

In order to **download the list** to a connected device, make sure that the Today List contains at least one patient and click 'Download'. **Note**: Please ensure that the device is connected to the PC, powered and in PC communications mode.

Change Password

Click 'Change Password' in the 'Customise' tab to change the current administrator password. Following window appears:

Enter the current administrator password (the default administrator password is 'Admin').

Click 'Ok' to continue.

If the entered password is correct, the password form will show two edit boxes. Enter the new password into both boxes and make sure they match.

Click 'Ok' to continue.

If the entered passwords are identical and the chosen password is valid, the password will be changed and the form will close automatically.

Note: The password is case-sensitive.

Note: If the administrator password is changed from default and you forgot the new password, please contact the Technical Support (see chapter 'Contacts') for details on how to reset the administrator password.

Exporting GDT

SPCS provides a GDT interface for interoperability with a practice electronic medical record system (PAS).

The PAS must be installed and configured to launch SPCS in the SPCS installation folder using the /GDT option.

Example:

If the default settings are used SPCS shall be started as follows:

c:\Program Files\Vyaire\Spirometry PC Software\Spirometry PC Software.exe **/GDT**.

When SPCS is first started after installation from your PAS the below dialogue box will be displayed.

ncoming Message C	:\Windows\temp		Browse
Dutgoing Message C	:\Windows\temp		Browse
Messages to GDT PMS		Message File naming	
GTD-ID GDT Name 8315 Receiver GDT-ID		Short name for this system	SPCS
8316 Sender GDT-ID		6228 - Result Table Te	ext Length 60 🚔
GTD-ID GDT Name 9218 GDT version V2. 9206 Character set CP	1 • 437 •		
Send link to report			
Report output directory(unc)	C:\Windows\temp		Browse
6304 File Content			
Report Filename	Create a unique filename on	each export	▼ PDF ▼
			Ok Abbrechen

The configuration for Incoming and Outgoing directories can be defined and must match the values set in the PAS.

Messages to GDT PMS

In this section the content of the GDT 8315 and 8316 segments are defined. (8315 refers to the SPCS system, 8316 refers to the PAS system).

Message File Naming

The "Short name for this system" is used to identify files intended for the software to read. Please refer to the GDT specification for further details on file naming conventions.

Result table text length

Please define the width of result table according to your requirements. The width is given in characters and does not include characters of the GDT header.

Message to GDT PMS (version 2.1 only)

If the SPCS system is configured for GDT version 2.1 additional options are available.

9218 Character set	Select the appropriate character set.
Send link to report	Together with the Outgoing GDT message a text report will be returned to the PAS. You may embed a link (path) to a WORD or PDF report in the GDT message. The default directory used to transmit this file to the PAS is the Windows Temp directory.
6304 File Content	Use this segment to include a fixed textu- al definition of the GDT message's content according to your requirements.
Report Filename	Use the drop down menu to select how the report is to be labelled and filed, ei- ther by Unique File name, or the file name using the Patients ID and exam date/ time. There is also the option to select whether the report is saved as a PDF or Word document.

NOTE:

Whenever started from the PAS, SPCS expects an incoming GDT message. If no such message is created, the following error message is displayed:

GDT settings can also be reconfigured after initial installation and configuration from the main tool bar in SPCS by selecting Data and Export Options.

After initial installation, to perform a spirometry test on the patient, the patient must be selected from the PAS. Once the patient is selected, to perform spirometry select Lung Function from the Practice software and SPCS will be launched for a test to be performed on the patient. The standard work flow is to perform relaxed spirometry followed by forced spirometry. However, if relaxed spirometry is not required, the Forced spirometry option can be selected for immediate testing.

After spirometry has been performed, the results should be saved in SPCS, and SPCS should be closed. If the user does not Save and Exit SPCS correctly, results will be exported to the Patient record. When SPCS is closed, the spirometry results will automatically be transferred to the patient's records in the Practice system along with any changes made to the patient demographics.

The indices (parameters) exported will be those that had previously been selected in SPCS using the Customise option and Spirometry Indices settings.

Data Upload from a device

SPCS interfaces to the device via a USB or serial port. It incorporates a database into which patient details can be entered and downloaded to the spirometry device or test results may be uploaded from the spirometry device to the PC.

Using SPCS and the spirometry device, live blows can be performed with the PC directly controlling the operation of the spirometry device. The results and graphs produced are displayed directly on the PC screen.

The spirometer is connected from the serial or USB port on the PC, to the USB port on the cradle using the USB cable provided with SPCS.

NOTE: Keep the PC and Monitor out of reach of the patient at all times.

It is recommended that whilst the unit is connected to a computer the spirometer remains in the cradle.

Symbols

In accordance with Directive 93/42/EEC

Consult the instructions for use

Manufacturer

Rx only

Date of Manufacture

the order of a physician.

Keep away from sunlight

Serial number

Reference number

Disposal of electronic devices in compliance with WEEE

Federal U.S. law restricts this device to sale by or on

Note: Information in this manual is subject to change without notice and does not represent commitment on the part of Vyaire Medical GmbH. The software may be used or copied only in accordance with the terms of that agreement. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording for any purpose without the written permission of Vyaire Medical GmbH.

International customers only

For all Sales Order processing for products and Spare parts, Service and Technical Support enquiries, please contact the following:

Vyaire Medical GmbH Customer Service & Support International Leibnizstrasse 7 97204 Hoechberg Germany

Customer Servi	ce Sales Enquiries:
Telephone:	0049 931 4972 670
Email:	micro.internationalsales@Vyaire.com

Factory Repair a	and Administration Enquiries:
Telephone:	0049 931 4972 867
Email:	support.admin.eu@Vyaire.com

Technical Suppo	ort Enquiries:
Telephone:	0049 931 4972 127
Email:	support.rt.eu@Vyaire.com

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Réf : ATS/ERS Task Force: Standardisation of Lung FunctionTesting. General considerations for lung function testing.M. Miller et al. Eur Resp J 2005:26. 153-161

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Vyaire Medical GmbH Internationale klantenservice en -ondersteuning Leibnizstrasse 7 97204 Hoechberg Duitsland
Vyaire Medical GmbH Leibnizstrasse 7 97204 Hoechberg Germany +49 931 4972-0 tel +49 931 4972-423 fax